

A-Clamp

“True Blue” A-Clamp
A NON-INVASIVE DEVICE
Safe - Strong - Lightweight

Occlude With Confidence

When it comes to safe and reliable occluding, you can depend on Dravon's "True Blue" A-Clamp. Smooth jaws are featured that allow for occluding 1-mm to 8-mm tubing without damage. A special nylon formulation makes the A-Clamp strong yet lightweight. Dravon's A-Clamp is a disposable that costs only a fraction of its metal counterpart.

Trusted By Blood Collection and Dialysis Centers

Don't mistake lesser quality imitations for Dravon's A-Clamp. Make sure you look for our name and phone number on the handle to be sure you are getting the clamp trusted by major medical facilities.

Caution

The A-Clamps will perform at maximum strength when used for the first time. With any plastic device, repeated exposure to heat and chemicals during the cleaning/disinfecting and/or sterilization procedure will cause a loss of clamp strength. You must remain alert for any weakness that may occur if this clamp is reused.

Cleaning/Disinfecting and/or Sterilization

A-Clamps can be cleaned using warm water and a detergent, such as Alconox®, mixed in a sonic cleaner. In addition, cold disinfectant solutions,



Jaws remain aligned when tubing is occluded.



Blood Collection






Dialysis

such as Cidex®, can be used to disinfect A-Clamps during a soak. Some solutions, such as bleach, can cause discoloration of the plastic and erosion of the metal rivet. Whenever possible, follow medical device cleaning protocols in your facility as well as warning labels on cleaning solutions to prevent or minimize damage during the process. After a cleaning/disinfecting procedure, the A-Clamp may be sterilized. Dravon maintains test data related to the autoclave and EtO method of sterilization on file.

Limited Warranty

Dravon warrants that each clamp is free from defects in material and workmanship. The A-Clamp should only be used to occlude PVC or silicone rubber tubing at operating temperatures above freezing. Please refer to the Directions for Use provided in each case of A-Clamps for additional information. Dravon does not warrant reused clamps.

Ordering Information

Part No. / Color	Item Description	Qty per case
A-110 BLUE	A-Clamp, Non-Sterile, Plastic Disposable 	100 each
A-120R RED	A-Clamp, Non-Sterile, Plastic Disposable 	100 each
A-120Y YELLOW	A-Clamp, Non-Sterile, Plastic Disposable 	100 each

Also available in starter packs of 25 each and in bulk-packed quantities of 500 each or more. For case pricing, call Dravon toll free at 1-800-654-1976 for a quotation, to ask questions or place an order.



Manufactured in the U.S.A.