QUICK START GUIDE

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STEP ONE

Download & Install Drivers

- 1a. Visit henryscheinequipmentcatalog.com/carina to download the CARINA™ 1.5 Digital Intraoral Sensor installer.
- 1b. Exit all open applications on your computer system.
- 1c. Locate the CARINA™ Software Installer (usually in your downloads folder) and double-click to run the installation wizard.
- 1d. Follow the on-screen prompts to install the driver and reboot your computer system when complete.*

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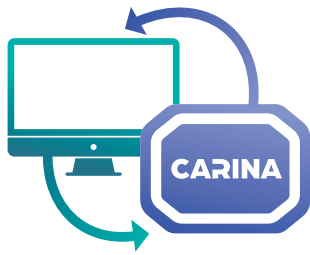
Scan code
to access
additional
support
materials.

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STEP TWO

Sync the CARINA™ 1.5 Digital Intraoral Sensor & Imaging Software.

- 2a. Connect your new CARINA™ Sensor to your computer via an available USB 2.0 or USB 3.0 port.
- 2b. Open your practice's imaging application.
- 2c. Locate your CARINA™ Sensor within the application and select the sensor.

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STEP THREE

Disinfect & Protect

Now that your imaging application and CARINA™ 1.5 Digital Intraoral Sensor are synced, you're almost ready to start making radiographs. Before snapping away, please wipe the CARINA™ 1.5 Digital Intraoral Sensor and its cable with a commercial disinfectant wipe and place the sensor inside a CARINA™ Sensor Barrier.

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STEP FOUR

Position & GO!

Use the included Positioning System to ensure precise placement of your CARINA™ 1.5 Digital Intraoral Sensor no matter how you're using it—vertically, horizontally, anterior, or posterior. Once you have the CARINA™ 1.5 Digital Intraoral Sensor properly positioned, you're all set to capture high quality X-ray images!

***Note: Rebooting is imperative for software to sync properly.**

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DIGITAL EASE. VALUABLE SOLUTIONS.

Exclusive to Henry Schein®. Contact your Henry Schein® Representative for more information.