

ONESTEP+ DRUG OF ABUSE RAPID URINE TEST CUP

**RELIABLE.
FAST.
CONVENIENT.**



Order today at HENRYSCHEIN.COM/MEDICAL




EXCLUSIVE BRANDS
& PRODUCTS



QUALITY YOU
CAN RELY ON



SMART
SAVINGS

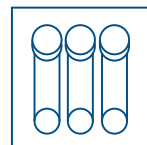

ONESTEP+ DRUG OF ABUSE RAPID URINE TEST CUP

Advantages

- CLIA waived
- Designed to detect a wide range of substances: AMP, BAR, BUP, BZO, COC, MDMA, EDDP, MTD, MET, MOP, OPI, OXY, PCP, PPX, TCA, and THC*
- Simple to use, no need to tilt cup to activate test
- Self-contained cup is ideal for sending preliminary positive specimens to the lab for confirmation
- Lid click feature ensures the cup is sealed properly, maintaining integrity of the test
- Results available in 5 minutes
- Temperature strip helps to monitor specimen tampering
- Adulteration strip offers initial screening and assists in monitoring any tampering of the specimen



*AMP-Amphetamine, BAR-Barbiturate, BUP-Buprenorphine, BZO-Benzodiazepines, COC-Cocaine, MDMA-Ecstasy, EDDP-Methadone Metabolite, MET-Methamphetamine, MTD-Methadone, MOP-Morphine, OPI-Opiates, OXY-Oxycodone, PCP-Phencyclidine, PPX-Propoxyphene, TCA-Tricyclic Antidepressants, THC-Marijuana

Test Procedure and Interpretation

1 Collect specimen and secure the lid.

2 Verify temperature is within an acceptable range. If applicable, read the adulteration strip pads against the provided color card.

3 Read results at 5 minutes.

Drug and Adulteration strip placement on the cup may vary depending on configuration.

RESULTS INTERPRETATION

NEGATIVE
C T C T

PRELIMINARY POSITIVE
C T

INVALID
C T C T

	Normal	Abnormal
OXI	Negative	
SG	1.005 1.015 1.025	1.000 >1.035
pH	4.0 6.0 7.0 8.0	2.0 3.0 9.0 12.0
NIT	Negative 20mg/dl	50mg/dl 100mg/dl
GLUT	Negative	
CREA	20mg/dl 100mg/dl	0mg/dl 10mg/dl

Henry Schein® OneStep+ Drug of Abuse Rapid Urine Test Cup 25/Box

Kit Contents: (25) OneStep+ Drug of Abuse Rapid Urine Test Cups, (2) Specimen Validity/Adulteration Strips (if applicable), (30) Disposable Gloves, (25) Security Seal Labels, (1) Instructions For use, (1) Procedure Card

Test Cup, 12 Panel—With Adulteration Strip..... (570-3927)
AMP1000, BAR300, BZO300, COC300, MDMA500, MET1000, MOP300, MTD300, OXY100, PCP25, TCA1000, THC50 (CR, SG, pH)

Test Cup, 14 Panel..... (570-3928)
AMP1000, BAR300, BUP10, BZO300, COC300, EDDP300, MDMA500, MET1000, MOP300, MTD300, OXY100, PCP25, TCA1000, THC50

Order today at [HENRYSCHIN.COM/MEDICAL](https://www.henryschein.com/MEDICAL)



EXCLUSIVE BRANDS
& PRODUCTS



QUALITY YOU
CAN RELY ON



SMART
SAVINGS

