

OneStep+ Ultra Flu

From A Name You Can Trust.

Simple to Use. Results in Minutes.

OneStep+ Ultra Flu Test Kit

The Henry Schein OneStep + Ultra Flu Test Kit provides superior rapid results at the point-of-care. Fast, easy, cost-effective so you can test and treat in one visit.

The One Step + Ultra Flu Test is intended for the qualitative detection of influenza type A and type B nucleoprotein antigens directly from nasal and nasopharyngeal swab specimens from patients with signs and symptoms of respiratory infection.

(570-1886)25/Box

Features & Benefits

- CLIA Waived
- Results in 10 minutes
- Sensitivity A/B 90.3%/88%
- Specificity A/B 96.7/99.2%
- For use with nasal and nasopharyngeal swab specimens



Test Procedure:


1. Insert the swab through the ridges into the liquid. Spin swab 10 times to mix the specimen.

NOTE: nasal swabs may not reach the bottom of the vial. ensure that the swab is fully immersed in the liquid when mixing.

2. Discard the swab. Insert Test Stick (arrows pointing downward) into the vial. Start timing.

3. Read test at 10 minutes. See Interpretation of Results.

NOTE: You may need to remove the Test Stick to read the test results.

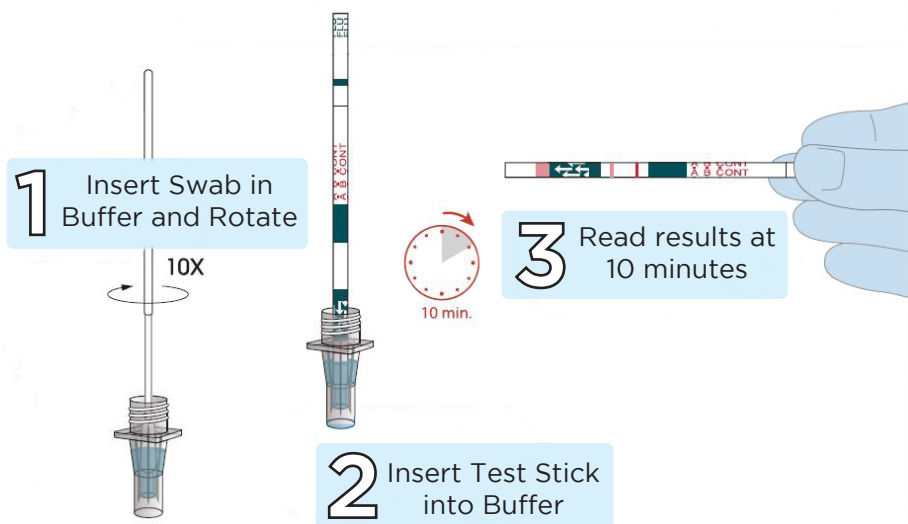

RELY ON US FOR QUALITY, SELECTION, PERFORMANCE, AND VALUE!