CHANNELS[®]

CHANNELS[®] ONE

EN: HEAT-TREATED NiTi RECIPROCATING FILES VARIABLE TAPER

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DIRECTIONS FOR USE

Indications for Use

Channels One Files are single use surgical instruments used for performing root canal treatment to mechanically shape and prepare the root canals during endodontic therapy or to remove the root canal obturating material when performing retreatment. The device is intended to be used sterile and single use only.

COMPOSITION

The instrument is made of a nickel-titanium blade, handle, the stop, and the color-coded band.

Contraindications

- Mechanically driven endodontic instruments should not be used in cases with very severe and sudden curvatures.
- This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

Warnings

- A rubber dam system should be used.
- Channels One Files that are non-sterile must be sterilized before patient use.
- Do not use if package is damaged.
- Channels One Files are for single use only, in order to avoid file separation.
- Rotary/Reciprocating files are sharp, and caution should be used if touching the blade directly.
- Used files should be disposed of in a Biohazard Sharps container in accordance with local regulations.
- This device is intended for single patient use only. The product and the packaging have not been designed or tested for reuse. The ability to effectively clean and re-sterilize this single use device and subsequent reuse may adversely affect the clinical performance, safety and/or sterility of the device.
- After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state, and federal laws and regulations.

Precautions for Use

As with all products, use carefully until you become proficient with use. Always determine working length using radiographs and/or apex locator to properly use rotary/reciprocating files. Important points to remember:

1. Use only in an electric motor and hand piece designed for the instruments.
2. Straight-line access is imperative for proper file use and endodontic treatment.
3. Do not force the files down canals, use minimal apical pressure.
4. Clean the flutes frequently and at least after removing the files from the canal.
5. Irrigate and lubricate the canal frequently throughout the procedure.
6. Take each file to length only one time and for no more than one second.
7. In apical areas and curved canals exercise caution.
8. Channels One Files are single patient use devices.
9. Once file is used do not reuse. If file is reused and used on a different patient infection can be introduced. Performance of the file can also be reduced.
10. When instrumenting the canal, do not over enlarge the coronal portion of the canal.
11. Too large a file taken to length increases the risk of canal transportation and file separation.
12. Channels One Files undergo our proprietary Annealed Heat Treatment (AHT) forming our branded NiTi which increases cyclic fatigue resistance and torque strength. With this proprietary processing, Channels One Files may be slightly curved. This is not a manufacturing defect. While the file can be easily straightened with your fingers, it is not necessary as once they are inside the canal, Channels One files will follow and conform to the natural canal anatomy and curvatures.

13. Do not exceed the handpiece recommended maximum torque or speed. Exceeding settings may cause the device to fail.

14. Do not use after the expiration date on the label.

Adverse Reactions

- Device fracture/breakage
- Infection – Do not use if package is damaged or open, due to risk of infection occurring.
- Complications usually associated with endodontic procedures including:
 - Pain
 - Instrument fracture/breakage
 - Soft tissue damage/bleeding

Safe Unwinding

- As a safety feature the files are designed to unwind. They may be used until the files unwind backwards.

Disinfecting:

- After each canal is fully shaped, rinse the canals for 1 minute with 17% Liquid EDTA to remove the canal Smear Layer.
- Rinse the canals for 5 minutes with 5% NaOCl to remove debris and bacteria.
- Rinse the canals for 1 minute with 17% Liquid EDTA to rinse out the 5% NaOCl.
- Rinse the canals for 5 minutes with 2% chlorhexidine or EDTA to kill bacteria.

Obturation of Canal Systems

- When using thermal carrier system use size verifiers to determine the proper sized carrier.
- When using a master gutta percha cone that matches the largest file taken to length, remember sometimes you may need to drop down in cone tip size if the corresponding gutta percha to your final rotary file does not go to length.

Storage

- Store at room temperature of 10°C~37.8°C, away from any sunlight. Recommended File Disposal Place used files in Biohazard Sharps container.

Sterilization

For files provided nonsterile only

- Files must be sterilized before use. ANSI/ADA Specification 28 recommends
- Scrub the instruments with soap and warm water.
- Rinse thoroughly with distilled or deionized water.
- Allow to air dry
- Place the instruments, unwrapped, in an autoclave tray.
- Use fresh distilled or deionized water.
- Steam Autoclave at 136° C (plus or minus 2° C) for 20 minutes.

Channels™ Size Selection

- If the #10 hand file was tight use the Channels One Small.
- If the #10 hand file was easy but the #15 hand file was tight use the Channels One Medium.

If both the #10 and #15 hand files were easy use the Channels One Large.

Channels Straight-Line Access

Create a glide path and determine the working length prior to Channel One file use by negotiating all root canals to their terminus with stainless steel #10 and #15 hand files and a lubricant.

Establish patency by taking a #10 K-File 1mm past the canal terminus, and at least a #15 K-File to the terminus.

Channels Canal Shaping and Cleaning

- The Channels One Files can only be used in a motor designed for WaveOne® instruments.
- Place the selected Channels file into the hand piece.
- With lubricant in the canal and light apical pressure, use a gentle inward pecking motion advancing the file 2-3 mm then lifting 1-2 mm. Keep repeating this motion to passively advance the Channels file until it does not easily progress.
- Remove the Channels One file from the canal, remove debris and inspect the file, irrigate, and recapitulate with a #10 hand file 1 mm past the canal terminus.
- Repeat steps 3 & 4 until the Channels One file is to the working length. If after repeated attempts the Channels One file does not seem to be advancing any further, drop down in Channels One file size and finish the canal.

Apically gauge the size of the foramen with a hand file the same tip size as the Channels One file taken to length. If the gauging hand file is a snug fit, the preparation is finished. If it is loose, use the next larger Channels One file to finish the preparation. Then obturate the canal.

Compatible Hand pieces

These files are used in endodontics for the removal of dentin and root canal shaping. It is compatible with the WaveOne Gold® reciprocating file system and must be used in the WaveOne Gold® motor and hand piece system using the WaveOne Gold® motor setting.

Hand Piece

Only use the Channels One file in same hand piece and motor that is designed for the WaveOne Gold® instrument using the WaveOne Gold® setting.

Electric Hand Piece

The Channels One file can only be used in an electric hand piece and motor designed for WaveOne Gold® instruments using the WaveOne Gold® setting. See manufacturer specifications.

Speed: 350 RPM (Reciprocating)

Torque: 4.0-5.2 Ncm/ 408-530 gcm

Symbol	Meaning (Standard, if Applicable)
	Manufacturer/ Legal Manufacturer (ISO 15223-1)
REF	Catalogue Number (ISO 15223-1)
LOT	Batch Code (ISO 15223-1)
	*Used-by Date (ISO 15223-1)
	Do Not Re-use (ISO 15223-1)
	Do not use if package is damaged (ISO 15223-1)
	Consult instructions for use (ISO 15223-1)
Rx Only	Caution: Federal law restricts this device to sale by or on the order of a “dentist/ Physician” licensed by the law of the State in which he/she practices to use or order the use of the device. (FDA 21 CFR’ Part 801.109 (b) (1))
STERILE R	*Sterile using irradiation (ISO 15223-1)
	Caution (ISO 15223-1)
	Temperature limit (ISO 15223-1)

* = Only present on sterile supplied product

For Dental Use Only
Rx Only
Sterilize Before Use

Distributed by:
Insight Endo
41 Weaver Road
Denver, PA17517 USA
REV. 2022/05
Made in the USA

Consult IFU before use
www.insightendo.com

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