

CRITERION™ NITRILE EXAM N100 GLOVES

**BUILT FOR
DURABILITY,
ENGINEERED
FOR COMFORT.**



SAME GREAT PRODUCTS, NEW PACKAGING



For more information about the full Criterion line of personal protection equipment scan here or visit HENRYSCHEINDENTAL.COM/CRITERION



CRITERION™ NITRILE EXAM N100 GLOVES

- Durable nitrile construction
- Chlorinated for reduced friction and easy donning
- Textured fingers provide enhanced grip
- Beaded cuff. 3.15-mil/0.08mm thickness at palm

BLUE COLOR

X-Small	100/box	111-8533
Small	100/box	111-8535
Medium	100/box	111-8536
Large	100/box	111-8537
X-Large	90/box	111-8539


CHEMOTHERAPY TESTED TO ASTM D6978 STANDARD

Caution: gloves used for protection against chemotherapy drug exposure must be selected specifically for the type of chemicals used. This glove has been tested for resistance to permeation of various chemotherapy drugs per ASTM D6978, "standard practice for assessment of resistance of medical gloves to permeation by chemotherapy drugs."

CHEMOTHERAPY DRUGS & CONCENTRATION*	MINIMUM BREAKTHROUGH DETECTION TIME
Cisplatin (1.0mg/mL)	>240min
Cyclophosphamide (Cytosan) (20mg/mL)	>240min
Dacarbazine (DTIC) (10.0mg/mL)	>240min
Doxorubicin Hydrochloride (2.0mg/mL)	>240min
Etoposide (Toposar) (1.0mg/mL)	>240min
5-Fluorouracil (50.0mg/mL)	>240min
Glutaraldehyde (80.0mg/mL)	>240min
Methotrexate (100.0mg/mL)	>240min
Paclitaxel (Taxol) (6.0mg/mL)	>240min
Vincristine Sulfate (1.0mg/mL)	>240min

*When chemotherapy drugs are present, gloves selection should be based on the specific types of drugs used. Users should review drug labeling or material safety data sheets for the drugs being used to determine an adequate level of protection.

SAME GREAT PRODUCTS, NEW PACKAGING

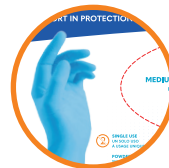

1. UPDATED PACKAGING

Makes it easy to know what's inside the box with attractive imagery and bold design elements.



2. INFO ZONE

Appears in the bottom right of all packaging. The icons indicate key product information: PPE standards, size, color and quantity.



3. REORDER PANEL

Helps to simplify the reordering process by clearly stating the full product name and reorder item number.