


## CRITERION™ NITRILE EXAM N200 GLOVES

### THE PERFECT PAIR: DURABILITY AND PERFORMANCE

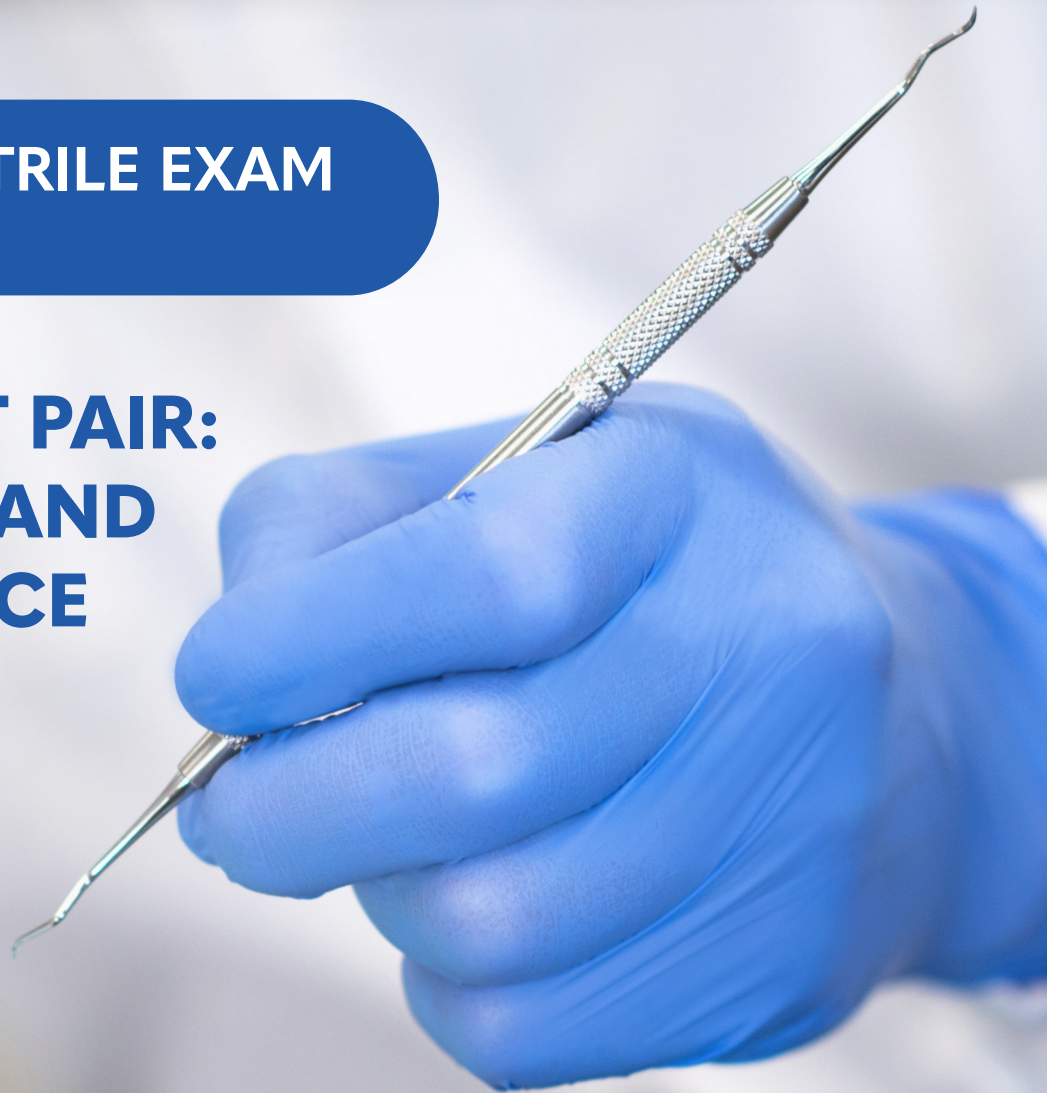


**SAME GREAT PRODUCTS, NEW PACKAGING**



For more information about the full Criterion line of personal protection equipment scan here or visit [HENRYSCHEINDENTAL.COM/CRITERION](https://HENRYSCHEINDENTAL.COM/CRITERION)

## CRITERION<sup>™</sup> NITRILE EXAM N200 GLOVES

- Durable nitrile construction, slightly thinner than the Criterion<sup>™</sup> Nitrile Exam N100 Gloves
- Chlorinated for reduced friction and easy donning
- Textured fingers provide enhanced grip
- Beaded cuff. 2.36-mil/0.06mm thickness at palm

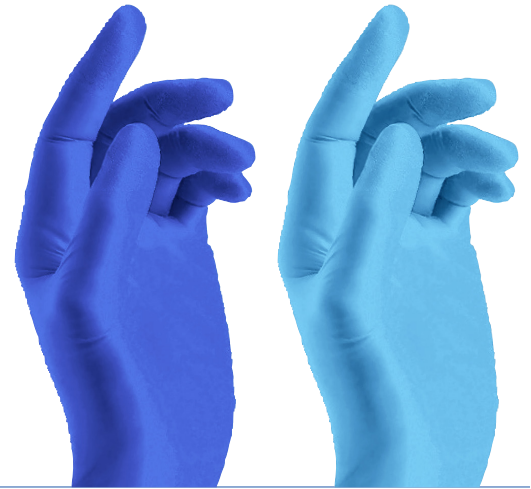

### PERIWINKLE BLUE

X-Small	200/box	900-7437
Small	200/box	900-7438
Medium	200/box	900-7439
Large	200/box	900-7440
X-Large	180/box	900-7441

### SKY BLUE

Small	200/box	570-2098
Medium	200/box	570-2099
Large	200/box	570-2100
X-Large	180/box	570-2101

### CHEMOTHERAPY TESTED TO ASTM D6978 STANDARD

**Caution:** gloves used for protection against chemotherapy drug exposure must be selected specifically for the type of chemicals used. This glove has been tested for resistance to permeation of various chemotherapy drugs per ASTM D6978, "standard practice for assessment of resistance of medical gloves to permeation by chemotherapy drugs."

CHEMOTHERAPY DRUGS & CONCENTRATION	MINIMUM BREAKTHROUGH DETECTION TIME
Carmustine (BCNU) (3.3mg/mL)	Do Not Use
Cisplatin (1.0mg/mL)	>240min
Cyclophosphamide (Cytoxan) (20mg/mL)	>240min
Dacarbazine (DTIC) (10.0mg/mL)	>240min
Doxorubicin Hydrochloride (2.0mg/mL)	>240min
Etoposide (Toposar) (20.0mg/mL)	>240min
Fluorouracil (50.0mg/mL)	>240min
Methotrexate (25mg/mL)	>240min
Mitomycin C (0.5mg/mL)	>240min
Paclitaxel (Taxol) (6.0mg/mL)	>240min
Thiotepa (10.0mg/mL)	Do Not Use
Vincristine Sulfate (1.0mg/mL)	>240min

**Warning: Do not use Carmustine (BCNU) (3.3mg/mL) and Thiotepa (10mg/mL)**  
When chemotherapy drugs are present, gloves selection should be based on the specific types of drugs used. Users should review drug labeling or material safety data sheets for the drugs being used to determine an adequate level of protection.

## SAME GREAT PRODUCTS, NEW PACKAGING



### 1. UPDATED PACKAGING

Makes it easy to know what's inside the box with attractive imagery and bold design elements.



### 2. INFO ZONE

Appears in the bottom right of all packaging. The icons indicate key product information: PPE standards, size, color and quantity.



### 3. REORDER PANEL

Helps to simplify the reordering process by clearly stating the full product name and reorder item number.