SINGLE USE AEROSOL EVACUATOR

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INSTRUCTIONS FOR USE

Read before use. Keep in safe place for future reference.

INDICATION FOR USE

This single-use HVE attachment is intended to be used during dental aerosol generating procedures for aerosol containment. It is designed for use by trained professionals in the field of dentistry.

⚠ Caution

- Professional use only
- Do not reuse
- Use personal protective wear during use.
- Do not modify or alter device.
- Never attempt to place or remove device while suction is running.

PRODUCT DESCRIPTION

1. Internal Wings
2. External Wings
3. Bumper
4. Suction Inlet
5. Valve Stem

OPERATION

To prevent upper lip from getting caught in suction inlet, do not allow patient to close mouth completely around device when HVE valve is on. If suction inlet gets blocked, turn off HVE valve and reposition device.

Installation (Fig. 1)

1. Insert device valve stem into HVE valve like a suction tip.
2. Rest valve in dock until ready to use.

Fig. 1

Placement

NOTE: The device can be adjusted by squeezing and holding the wings for 3 seconds in the desired position.

1. Ensure HVE suction system is ON and HVE valve is OFF.
2. Have patient open mouth and insert one side of device by angling the internal and external wings into the patient's mouth (Fig. 2).
3. Rotate the device to insert internal and external wings on other side. (Fig. 3).

Fig. 2

Fig. 3

4. Seat device by pulling it forward and down to position bumper in front of lower incisor gums and so external wings rest on bottom lip. Ensure HVE line is directly above midline of upper body and not angled in patient's mouth (Fig. 4).

Note: If patient purses or tightens lips, device may push against lower gums. For improved comfort, place a folded 2x2 cotton gauze pad/sponge between device and gums (Fig. 5). The evacuator can be used in conjunction with a saliva ejector.

Fig. 4

Fig. 5

Operation

1. Tilt patient's head back slightly so that device sits in line with HVE valve to avoid bending at valve connection point. Patients may hold HVE line if they feel more comfortable (Fig. 6).
2. Turn HVE valve ON.

Fig. 6

DISPOSAL

The Single Use Aerosol Evacuator is meant to be used only once and should not be sterilized or reused. Properly discard the evacuator in a medical waste container to prevent transmission of infectious pathogens.

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