

Put your practice ONE STEP ahead of today's frantic pace with



HENRY SCHEIN®

OneStep Pro+

Strep A Cassette

Count on it for...

EASE OF USE

- CLIA-waived
- Unique device design:
All in one extraction,
incubation and test

RELIABILITY

- Proven patented technology
- 98.7% specificity; 96.2%
sensitivity

CONVENIENCE

- Room temperature storage
- Kit includes external controls
- Accurate results in
5-10 minutes



 HENRY SCHEIN®
MEDICAL

Item # 9007570
OneStep Pro + Strep A Cassette
25 Cassettes

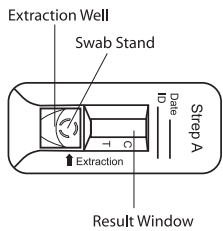
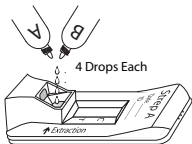
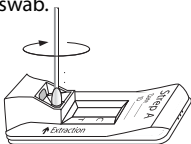
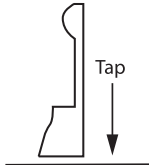
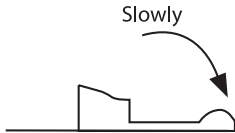
To Order: 1-800-772-4346 8am-9pm (et)
To Fax: 1-800-329-9109 24 Hours
www.henryschein.com/medical

Strep A Test

STREP A TEST COMPARISON CHART

Product Name	Henry Schein OneStep Pro + Strep A Cassette	Quidel QuickVue In-Line Strep A Test	Other Brands Strep A test
Sensitivity	96%	92%	96%
Specificity	99%	99%	96%
CLIA Complexity	Waived	Waived	Waived
Format	25 Cassettes	25 Cassettes	25 Dipsticks
External Controls	Pos + Neg Controls (included)	Pos + Neg Controls (included)	Pos + Neg Controls (included)
Room Temperature Storage	Yes	Yes	Yes
Extraction Reagents	2 (Extraction A and B)	2 (Reagents in 1 capsule method)	2 (Extraction A and B)
Swabs	25	25	25

See package insert for details

TEST DEVICE	PROCEDURE
 <p>Extraction Well Swab Stand Date Strep A Extraction Result Window</p>	<ol style="list-style-type: none"> Add 4 drops each of Reagent A and Reagent B into extraction well. Place specimen swab into extraction well and spin 5 times to mix specimen. Let incubate for 1-2 minutes and spin 5 more times. Remove swab. Raise the device upright, tap device on a flat surface to ensure the specimen flows into the hole. Slowly lay the device back down onto flat surface and read results after 5 minutes.    

Item # 9007570
OneStep Pro + Strep A Cassette
25 Cassettes

To Order: 1-800-772-4346 8am-9pm (et)
To Fax: 1-800-329-9109 24 Hours
www.henryschein.com/medical

 **HENRY SCHEIN®**
MEDICAL